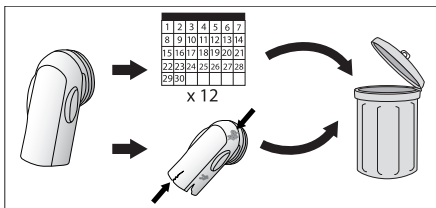
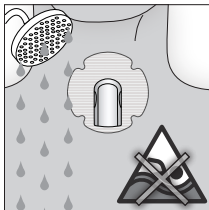
# PROVOX<sup>®</sup> Shower Life


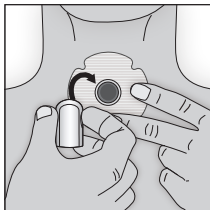
1.


2.

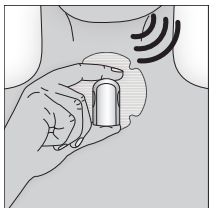
3.

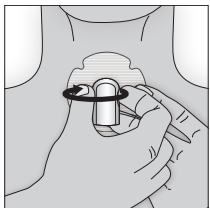
4.

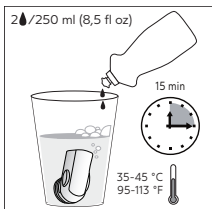
5.

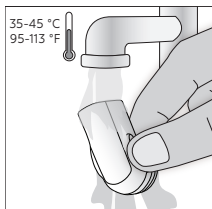
6.

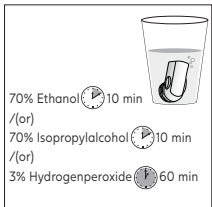7.

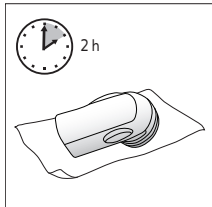8.

9.

## **Disclaimer**

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

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# ENGLISH

## 1. Intended use

Provox Life™ Shower is a single patient use device intended to be placed into a Provox Life™ attachment to avoid water from entering the tracheostoma during showering.

## 2. Contraindications

The device should only be used in accordance with the Instructions for Use. Patients without the physical, cognitive, or mental ability required to attach, remove or operate the device themselves, should not use the device independently and should only use it if they are under sufficient supervision of a clinician or a trained caregiver.

## 3. Warnings

- Do not reuse the device between patients as it may cause cross-contamination. The device is intended for single patient use only.
- Do not squeeze, or apply excessive force during attachment or when the device is in place. Excessive force can result in the device being pushed through the adhesive coupling, and may obstruct breathing and block airways.
- Do not use non-compatible or non-original devices as it may cause personal injury or damage the device.
- Do not dismantle or alter the device as it may cause personal injury or damage the device.
- Do not use a dirty or damaged device as it may cause infection (Fig. 1).

- Do not use the device when swimming/bathing as it may lead to aspiration of water which may result in aspiration pneumonia (Fig. 2).

## **4. Instructions for use**

### **4.1 Operating instructions**

Remove Provox Life™ HME and attach Provox Life™ Shower before showering (Fig. 3). Occlude Provox Life™ Shower opening to allow speech (Fig. 4). To remove, hold the device and pull sideways until the device disconnects (Fig. 5).

### **4.2 Cleaning and disinfection**

Clean and disinfect Provox Life™ Shower before each use. Only use drinking water to clean and rinse the device.

1. Place Provox Life™ Shower in water-based mild soap solution for 15 minutes (Fig 6).
2. Rinse Provox Life™ Shower in water (Fig 7).
3. Disinfect Provox Life™ Shower (Fig. 8) with one of the following methods: Ethanol 70% for 10 minutes or Isopropylalcohol 70% for 10 minutes or Hydrogenperoxide 3% for 60 minutes.
4. Let Provox Life™ Shower dry at least for 2 hours or until it looks dry (Fig. 9).

## **5. Storage instructions**

Store Provox Life™ Shower in a clean and dry container at room temperature. Protect from direct sunlight.

## **6. Device lifetime and disposal**

Provox Life™ Shower may be used for a maximum period of 12 months (Fig. 1). If the device shows any signs of damage, it shall be replaced earlier. Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

## **7. Compatible products**

Provox Life™ Shower is compatible with Provox Life™ attachments.

## **8. Reporting**

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patients resides.

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© Coloplast A/S. All rights reserved.
trademarks of Coloplast A/S.
Atos Medical AB