PROVOX[®]

Vega[™] Plug


MD


CE 0413





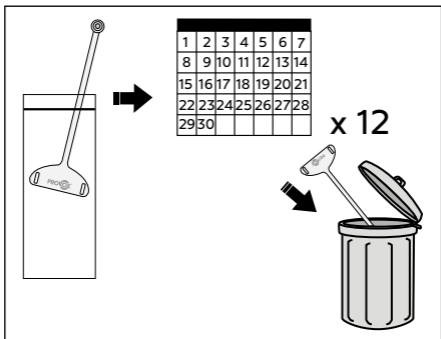
42° C
108° F
RT
2° C
36° F

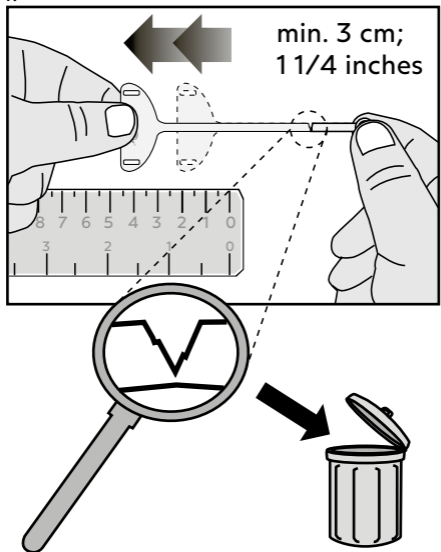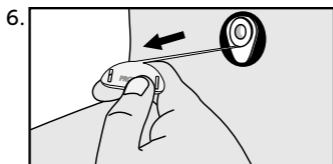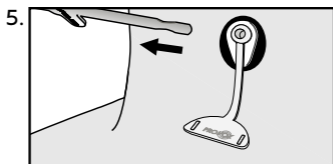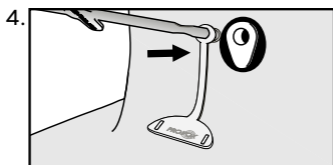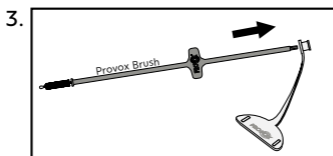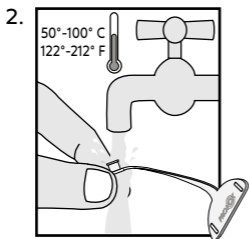
1.

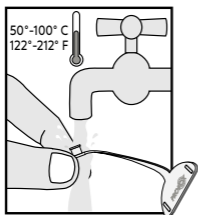
7.


8.

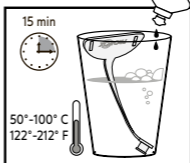
2 /250 ml (8,5 fl oz)

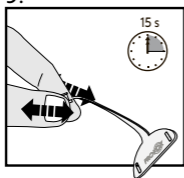
9.

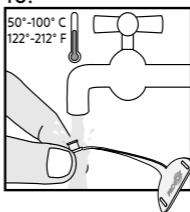
10.

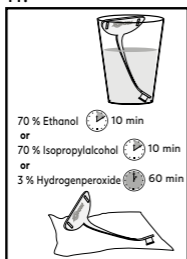11.

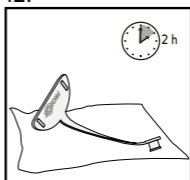12.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. Provox® Vega™ is a trademark of Atos Medical AB. For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents

**Compatible Products; Kompatible Produkte;
Compatibele producten; Produits compatibles ;
Prodotti compatibili; Productos compatibles;
Produtos compatíveis; Kompatibla produkter;
Kompatible produkter; Kompatible produkter;
Yhteensopivat tuotteet; Samhæfar vörur;
Ühilduvad tooted; Suderinami gaminiai;
Kompatibilní produkty; Kompatibilis termékek;
Kompatibilné produkty; Združljivi izdelki;
Zgodne produkty; Produe compatibile;
Kompatibilni proizvodi; Kompatibilni proizvodi;
Συμβατά προϊόντα; Совместимые изделия;
Uyumlu Ürünler; Համատեղելի արտադրանքներ;
Produk yang Serasi; 互換性のある製品;
호환 가능한 제품; 相容產品; מוצרים תואמים;
المنتجات المتوافقة**

Provox Vega
Provox Vega XtraSeal

Contents

EN - ENGLISH.....	7
DE - DEUTSCH.....	9
NL - NEDERLANDS.....	11
FR - FRANÇAIS.....	13
IT - ITALIANO	15
ES - ESPAÑOL.....	17
PT - PORTUGUÊS.....	19
SV - SVENSKA	21
DA - DANSK.....	23
NO - NORSK.....	25
FI - SUOMI.....	27
IS - ÍSLENSKA	29
ET - EESTI.....	31
LT - LIETUVIŲ KALBA	33
CS - ČESKY	35
HU - MAGYAR	37
SK - SLOVENČINA.....	39
SL - SLOVENŠČINA	41
PL - POLSKI.....	43
RO - ROMÂNĂ	45
HR - HRVATSKI.....	47
SR - SRPSKI	49
EL - ΕΛΛΗΝΙΚΑ	51
RU - РУССКИЙ.....	53
TR - TÜRKÇE.....	55
HY - ՀԱՅԵՐԵՆ	57
MS - BAHASA MELAYU	59
JA - 日本語	61
KO - 한국어.....	63
ZHTW - 繁體中文.....	65
HE - עברית	68
AR - العربية	70

1. Descriptive Information

1.1 Intended use

Provox Vega Plug is intended to seal the inner lumen of Provox Vega voice prosthesis and therefore stop leakage of both air and fluids through the voice prosthesis.

1.2 Description of the device

Provox Vega Plug is a first-aid tool for temporarily stopping leakage through a Provox Vega voice prosthesis. The device is inserted into the opening of the Provox Vega voice prosthesis and hence blocking any leakage through the valve.

1.3 Precaution

Provox Vega Plug is intended for single patient use. Reuse in another patient may cause cross contamination.

2. Instructions for use

2.1 Operation instruction

1. Check device integrity prior to each use by pulling the strap at least 3 cm (1 1/4 inches) and check for damages. If the device shows any signs of damage the device shall be discarded (Fig. 1).
2. Rinse the device in water (Fig. 2).
3. Insert Provox Vega Plug into the lumen of the prosthesis by using Provox Brush (Fig. 3-5).
4. Ensure proper position by gently pulling the strap (Fig. 6).

2.2 Cleaning and disinfection

Clean and disinfect the device after each use. Only use drinking water to clean and rinse the device.

1. Rinse the device in water (Fig. 7).
2. Place the device in water-based mild soap solution for 15 minutes (Fig. 8).
3. Rub the device between your fingers for 15 seconds (Fig. 9).
4. Rinse the device in water (Fig. 10).

5. Disinfect the device (Fig. 11) with one of the following methods:
 - Ethanol 70% for 10 minutes
 - Isopropylalcohol 70% for 10 minutes
 - Hydrogenperoxide 3% for 60 minutes
6. Let the device dry at least for 2 hours or until it looks dry (Fig. 12).

CAUTION: DO NOT clean or disinfect by any other method as this might cause device damage and injury.

If Provox Vega Plug looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use.

Do not use Provox Vega Plug until it is completely dry. Inhalation of disinfectant fumes can cause severe coughing and irritation of the airways.

Provox Vega Plug shall be cleaned and disinfected before each use to avoid risk of contamination. Provox Vega Plug is for single patient use only, reuse between patients may cause cross contamination.

During hospitalization it is important to clean and disinfect Provox Vega Plug immediately after use and again just before use due to an increased risk of infection and device contamination. Use sterile water for rinsing, rather than drinking water.

2.3 Device lifetime and disposal

Provox Vega Plug may be used for a maximum period of 12 months. If the device shows any signs of damage it shall be replaced earlier. Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

2.4 Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Atos Medical AB, Sweden.